



wet - moist - dry
wound

DERMANET® GTL
silicone wound contact layer



dry wound

DERMANET®
wound contact layer



moist wound

DERMANET® AG+
antimicrobial barrier wound contact layer



Dermanet® Wound Contact Layers are the perfect option for wound protection. Original Dermanet® and Dermanet® AG+, containing silver ions, are made from a proprietary blend of high-density polyethylene.

And now introducing **Dermanet® GTL**, with the strength of Dermanet® and the softness of silicone.

DERMANET® GTL

silicone wound contact layer



features and benefits:

- Helps to facilitate fluid transfer and provide fixation and protection to the wound bed during dressing changes
- Ideal for use in traditional wound applications
- Easy 1-2-3 Release Liners on larger sizes
- Rounded edges to reduce "roll-up"
- Open mesh design is coated with a soft silicone adhesive
- Dressings may be removed without causing trauma to the wound or to the surrounding skin

Product #	Size	Qty/Bx	Qty/Cs	*HCPC
46-G23	2" x 3"	25	100	A6206
46-G34	3" x 4"	25	100	A6206
46-G47	4" x 7"	25	100	A6207
46-G810	8" x 10"	25	100	A6208



DERMANET®

wound contact layer



features and benefits:

- Excellent non-adherent properties without compromising the flow of drainage from the wound
- Inert (totally unreactive), non-linting
- Adequate tensile strength without sacrificing the conformability and softness in a wound dressing
- Air and fluid permeable
- Place directly on wound to protect wound surface during healing
- Shields fragile granulation tissue from disturbance caused by secondary dressings
- May be used dry or coated with topical ointment
- Ideal for lining deep wounds before the application of packing materials or under gauze
- Reduces trauma during wound dressing removal

DERMANET® AG+ FAMILY

antimicrobial barrier adhesive wound dressing



Dermanet® Ag+ is a combination of Dermanet® wound contact layer and Algidex Ag® Silver technology. It helps to reduce trauma and pain that can be caused during dressing changes. Algidex Ag decreases odor, and creates a moist environment conducive to healing.

Product #	Size	Qty/Bx	Qty/Cs	*HCPC
46-145	3" x 3"	25	50	A6206
46-143	5" x 4"	25	50	A6207
46-142	8" x 10"	25	50	A6208
46-141	20" x 15"	25	50	A6208
46-140	24" x 36"	n/a	50	A6208
46-144	6" x 72" roll	n/a	25	A6208

indications:

Burns
Chronic wounds

features and benefits:

- Provides quick and sustained antimicrobial barrier against a broad spectrum of external wound pathogens including methicillin-resistant Staphylococcus aureus (MRSA), Staphylococcus aureus, Pseudomonas aeruginosa, E. Coli *
- A unique combination of silver, alginate, maltodextrin, and Dermanet®
- Ionic silver contacts wound pathogens
- Does NOT require activation or the use of sterile distilled water
- Easy application and removal
- Will not adhere to wound upon removal
- Protects fragile granulation tissue

indications:

Pressure ulcers	1st and 2nd degree burns
Venous ulcers	Donor sites
Dermal ulcers	Graft sites
Diabetic ulcers	Abdominal wounds
Dermal lesions	Superficial wounds
Burn injuries	Lacerations, cuts, and abrasions



DERMANET® AG+

antimicrobial barrier adhesive wound dressing



dry wound

Product #	Size	Qty/Bx	Qty/Cs	*HCPC
46-DN22	2" x 2"	5	50	A6206
46-DN44	4" x 4"	5	50	A6206
46-DN48	4" x 8"	5	50	A6207
46-DN66	6" x 6"	5	25	A6207
46-DN88	8" x 8"	5	25	A6208
46-DN816	8" x 16"	5	25	A6208




DERMANET® AG+ BORDER

antimicrobial barrier adhesive wound dressing with border



dry wound

Product #	Size	Qty/Bx	Qty/Cs	*HCPC
46-DB22	2.5" x 2.75"	25	100	A6206
46-DB44	4" x 4"	25	100	A6206
46-DB46	4" x 6"	25	100	A6206
46-DB48	4" x 8"	25	100	A6207
46-DB410	4" x 10"	25	100	A6207
46-DB414	4" x 14"	25	100	A6208

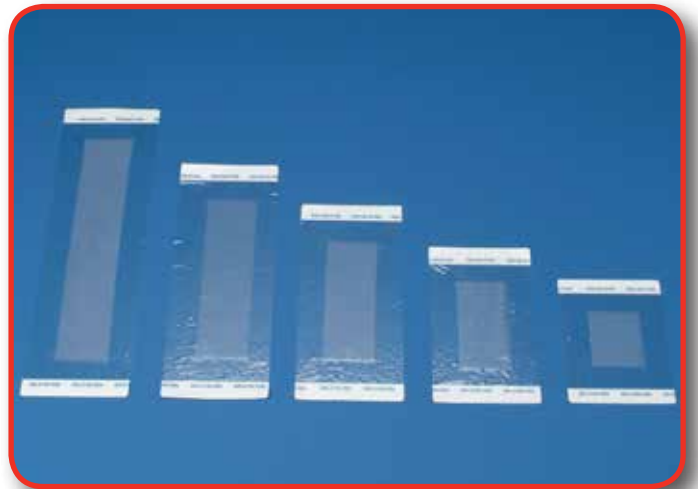

DERMANET® AG+ PLUS

antimicrobial barrier absorbant and adhesive wound dressing with border



wet - moist wound

Product #	Size	Qty/Bx	Qty/Cs	*HCPC
46-A44	4" x 4"	25	100	A6206
46-A46	4" x 6"	25	100	A6206
46-A48	4" x 8"	25	100	A6207
46-A410	4" x 10"	25	100	A6207
46-A414	4" x 14"	25	100	A6208




dermanet® & dermanet® ag+ application:

1. Thoroughly cleanse wound.
2. Remove product from backing material or amount needed from roll.
3. Place product over shallow wounds or pack into deeper wounds.
4. Cover with appropriate secondary dressing (e.g. gauze, Multipad™, Sofsorb®, Polyderm™ dressings, etc...) based on amount of drainage.
5. Secure dressing in place as needed with tape, roll gauze, or Stretch Net™ dressing.
6. Dermanet® Ag+ can be worn up to seven days or until the secondary dressing is saturated and requires changing.

dermanet® ag+ border & dermanet® ag+ plus application:

1. Clean and thoroughly dry site.
2. Peel release liner from center of dressing while aligning the Dermanet® Ag+ dressing center over the site.
3. Press dressing into place.
4. Remove the "Top Clear Carrier Film" by lifting away at the tab.

survival curves (ag)*:

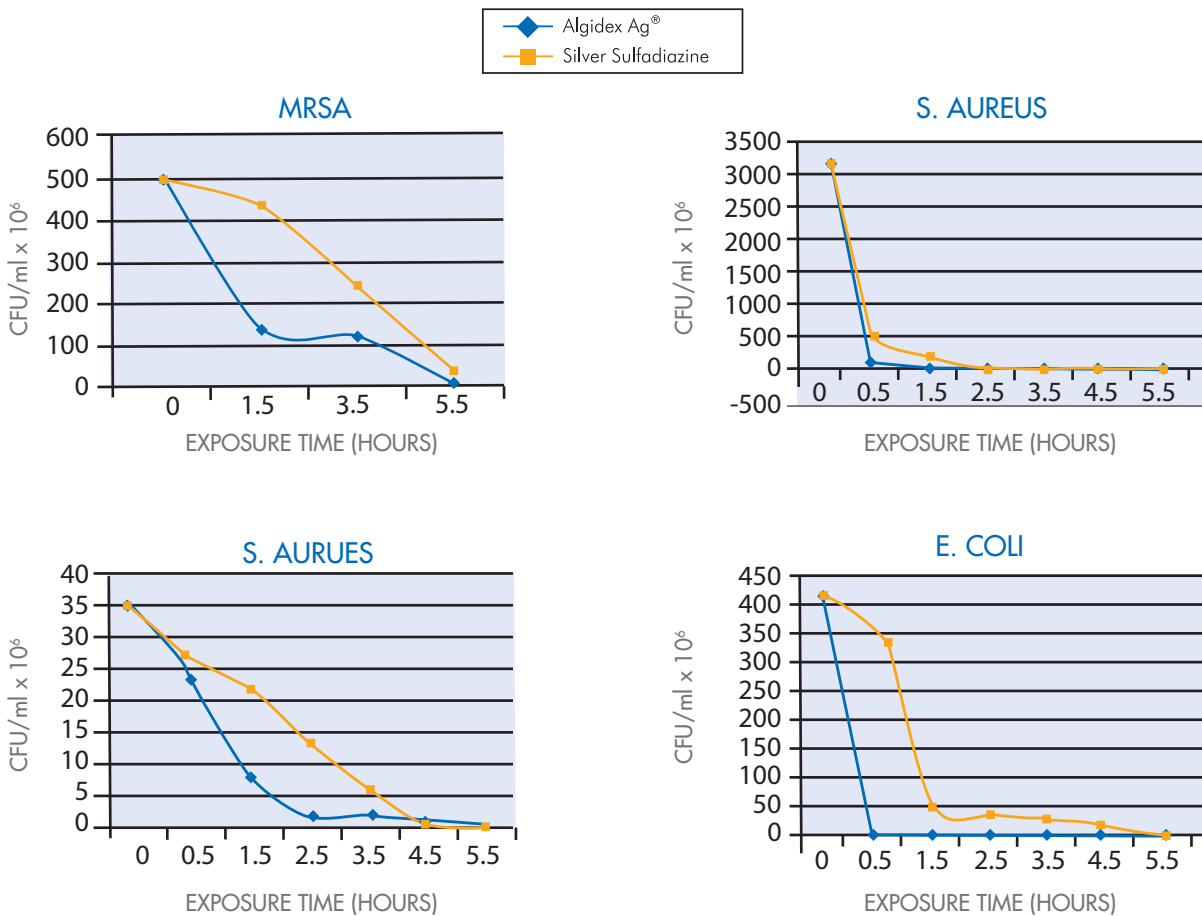

* Any coding information provided by DeRoyal is only a recommendation provided for the convenience of and as a resource to healthcare professionals. These recommendations are based upon private and public information including L-Code descriptors, ICD-10 diagnostic codes, Medicare PDAC information and information from experienced consultants and DMEPOS suppliers. DeRoyal does not guarantee or imply claim reimbursement or coverage for any specific DeRoyal product or in any specific amount. For more information, please visit our website.

DeRoyal®, DeRoyal logo, Improving Care. Improving Business.®, Dermanet®, Algidex Ag®, and Sofsorb® are registered trademarks of DeRoyal Industries, Inc.

Multipad™, Polyderm™, and StretchNet™ are trademarks of DeRoyal Industries, Inc.

© 2017, DeRoyal. All rights reserved.

*data on file