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DOUBLE-LOCK SECURITY CUFFS W/ QUICK RELEASE BUCKLE

	SINGLE PATIENT USE
	NON-STERILE
	NOT MADE WITH NATURAL RUBBER LATEX
	FEDERAL U.S.A. LAW RESTRICTS THIS DEVICE TO SALE OR USE BY OR ON THE ORDER OF A PHYSICIAN OR PROPERLY LICENSED PRACTITIONER.

IMPORTANT INFORMATION

Please read all instructions, warnings, and precautions before use. Correct application is essential for proper functioning of the product. Use only on the person it was provided to by a healthcare professional and only for the use it was intended.

INTENDED USE

The DeRoyal® Double-Strap Security Cuffs are intended to be prescribed by a properly licensed practitioner to protect the patient from harming themselves, other patients, family members and/or staff.

CONTRAINDICATIONS

Never use on a patient if an IV or wound site could be compromised by the device.

WARNINGS

- Improper application or use of any restraint may result in serious injury or death.
- Secure straps to a portion of the bed or gurney that moves with the patient. Do not secure straps to bed or gurney side rails.
- Keep straps out of patient's reach. Patient should not be able to remove device.
- Make sure bed or gurney side rails are up while product is in use.
- Check the patient regularly to ensure circulation is not compromised.

CAUTIONS

This product is to be fitted initially by a physician (or properly licensed practitioner) who is familiar with the purpose for which it is intended. The physician or practitioner is responsible for providing wearing instructions and precautions to other healthcare practitioners or healthcare providers involved in the patient's care as well as the patient. Consult your physician (or properly licensed practitioner) immediately if you experience sensation changes, unusual reactions, swelling or increased pain while using this product. Follow the instructions of your physician (or properly licensed practitioner) for length and duration of use.

Be sure to follow your facility's policies and guidelines for frequency of patient monitoring.

INSTRUCTIONS FOR USE

- Secure the cuffs to the bed or gurney. Release the female end of the buckle and wrap its strap around the anchor.
- Pull the female portion through the loop and connect the male and female ends of the buckle.
- Pull the excess strap to tighten. **NOTE:** Secure excess strap from patient's reach.
- Once all security cuffs are applied to the bed or gurney, secure the cuffs on the patient.
- Wrap the neoprene cuff around the patient's wrist or ankle with the

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loose neoprene side first and then follow with the neoprene side with hook. Secure the loop side of the cuff and follow with the other hook side of the cuff.

SIZING

Cuff Dimensions	Strap Length
14"L x 3"W (35 x 7 cm)	49" (125 cm)
16"L x 3"W (40 x 7 cm)	49" (125 cm)
8"L x 3"W (20 x 7 cm)	49" (125 cm)

CLEANING AND/OR MAINTENANCE

To clean, hand wash with a mild fabric detergent in warm water. Do not bleach. Do not iron. Lay flat and open and allow to air dry. Inspect the product after washing to ensure it is intact. If the product is damaged, do not use it and replace it with a new one.

STORAGE AND TRANSPORT CONDITIONS

	KEEP DRY
	KEEP AWAY FROM SUNLIGHT

In addition to the competent authority in the country where the patient resides, serious incidents must be reported to DeRoyal Industries, Inc.

WARRANTY

DeRoyal® products are warranted for one hundred twenty (120) days from the date of shipment from DeRoyal as to product quality and workmanship. **DEROYAL'S WRITTEN WARRANTIES ARE GIVEN IN LIEU OF ANY IMPLIED WARRANTIES, INCLUDING WARRANTIES OF MERCHANTABILITY OR FITNESS FOR A PARTICULAR PURPOSE.**

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