



Improving Care. Improving Business.®

DEROYAL® SPORTS ORTHOSIS

	SINGLE PATIENT USE
	NON-STERILE
	NOT MADE WITH NATURAL RUBBER LATEX
RX ONLY	FEDERAL U.S.A. LAW RESTRICTS THIS DEVICE TO SALE OR USE BY OR ON THE ORDER OF A PHYSICIAN OR PROPERLY LICENSED PRACTITIONER.

IMPORTANT INFORMATION

Please read all instructions, warnings and cautions before use. Correct application is essential for proper product function and to reduce the risk of injury or re-injury inherent with the use of any brace. Use only on the person it was provided to by a healthcare professional and only for the use it was intended.

INTENDED USE

The DeRoyal® Sports Orthosis is for chronic instabilities of the ankle, prophylactic talocrural support, subtalar joint sprain, syndesmosis and prevention and treatment of acute ankle sprains.

CAUTIONS

- A physician or properly licensed practitioner (a “prescriber”) who is familiar with the use and purpose of this brace must fit it to the user. The prescriber has a duty to provide wearing instructions and risks related to the use of this brace to other healthcare practitioners treating the users and the users themselves, including duration of use. The instructions provided in this sheet do not supersede hospital protocol or direct orders of the prescriber. Use only as directed.
- Prolonged usage can result in muscle atrophy in the user. The user may require (1) a weaning period from the brace to build muscle strength (2) regular physical therapy exercises to maintain the strength of the ankle (3) a limitation on duration of use or (4) some combination of these.
- Inspect brace for damaged or missing components before use.
- Discontinue use and consult your prescriber if the brace or its components break or become damaged.
- Inspect the fit of the brace on the patient to ensure fit is proper. **DO NOT OVERTIGHTEN.** Check regularly to ensure circulation is not compromised.
- Consult your prescriber immediately if you experience sensation changes, unusual reactions, swelling or increased pain while using this brace. Discontinue use if pressure injuries develop.
- Take special care if the user is diabetic or has poor circulation. These users may have decreased skin sensitivity and are at greater risk for poor peripheral circulation and pressure injuries.



SIZE

X-Small
Small
Medium
Large
X-Large
XX-Large

LENGTH ADJUSTMENT*

10” - 11”
11” - 12”
12” - 13”
13” - 14”
14” - 15”
15” - 16”


* Measure circumference from the base of heel around the ankle.

INSTRUCTIONS FOR USE

NOTE: A sock should be worn under this brace.

APPLICATION

- Place foot with the heel to the back of the brace and position tongue between foot and laces. For proper fitting, apply brace with the ankle positioned at a 90° angle.
- Lace up ankle brace from the bottom to top and tie laces together.
- Beginning with inside (medial) strap, bring strap across the top (dorsal) aspect of the foot. Wrap strap under the heel and back around attaching to side of support. Repeat in the opposite direction for outside (lateral) strap.
- Wrap wide elastic closure around leg and secure with label in front. Do not restrict circulation. Readjust as needed for a secure but comfortable fit.

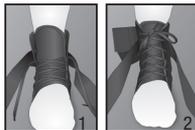

CARE INSTRUCTIONS

Hand or machine-wash on gentle cycle with cold water and mild detergent. Air dry.





Improving Care. Improving Business.®

STORAGE AND TRANSPORT CONDITIONS

	KEEP DRY
	KEEP AWAY FROM SUNLIGHT

WARRANTY

DEROYAL® PRODUCTS ARE WARRANTED FOR NINETY (90) DAYS FROM THE DATE OF SHIPMENT FROM DEROYAL AS TO PRODUCT QUALITY AND WORKMANSHIP. **DEROYAL'S WRITTEN WARRANTIES ARE GIVEN IN LIEU OF ANY IMPLIED WARRANTIES, INCLUDING WARRANTIES OF MERCHANTABILITY OR FITNESS FOR A PARTICULAR PURPOSE.**



Improving Care. Improving Business.®



Manufactured by:
DeRoyal Industries, Inc.
200 DeBusk Lane
Powell, TN 37849 USA
888.938.7828 or (001) 865.938.7828
www.deroyal.com

Part#0-1844 Revised 12/2021

©2021 DeRoyal Industries, Inc.

All Rights Reserved. DeRoyal, the DeRoyal logo, and Improving Care. Improving Business. are trademarks or registered trademarks of DeRoyal Industries, Inc.