



DEROYAL® FALL MAT (M10-020)

	REUSABLE
	NON-STERILE
	NOT MADE WITH NATURAL RUBBER LATEX

IMPORTANT INFORMATION

Please read all instructions, warnings and cautions before use. Correct use of this product is essential for proper functioning of product.

INTENDED USE

The DeRoyal® Fall Mat is intended to help provide protection to the patient in the event of a fall from a bed, wheelchair, stretcher, or recliner.


WARNINGS

- **DO NOT** fold the mat.
- **DO NOT** immerse the mat in any liquid cleaning solution.
- **DO NOT** place the mat over wet surfaces.
- **DO NOT** expose the mat to high temperature sanitizers.
- Inform the patient and staff to use caution when stepping on or off the mat to avoid tripping.
- Ensure product is dry and free of spills and/or moisture before standing or stepping on mat. Liquids on or under the mat may cause a slipping hazard.

THE MANUFACTURER DOES NOT CLAIM THAT THIS DEVICE WILL PROTECT ANY AND/OR ALL INJURIES SUSTAINED IN THE EVENT OF A FALL.

CAUTIONS

- This product is to be used by a clinician (or properly licensed practitioner) who is familiar with the purpose for which it is intended. The clinician is responsible for providing instructions and precautions to other healthcare practitioners or healthcare providers involved in the patient's care as well as the patient. Follow the instructions of a provider (or properly licensed practitioner) for length and duration of use.

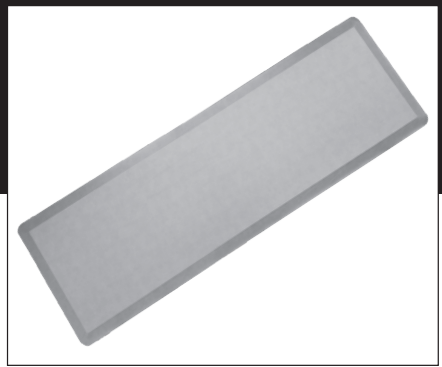

INSTRUCTIONS FOR USE

1. Place a fall mat on the floor beside the bed. If using two mats, place one on either side of the bed.
2. Ensure the dark gray, textured side of the mat is facing the floor.
3. Make sure the fall mats are placed adjacent to the length of the bed so there is no exposed floor between the bed and the mat.
4. Make sure the mat lays flat on the floor and is not bunched or raised.

SIZING: Universal 1" x 24" x 70"

CLEANING INSTRUCTIONS

Wipe clean with disinfectant or anti-bacterial cleaning solution. Ensure top and bottom of mat are fully dry before use. **DO NOT** use abrasive cleaning agents.

STORAGE AND TRANSPORT CONDITIONS

	KEEP DRY
	KEEP AWAY FROM SUNLIGHT

In addition to the competent authority in the country where the patient resides, serious incidents must be reported to DeRoyal Industries, Inc.

WARRANTY

DeRoyal® products are warranted for ninety (90) days from the date of shipment from DeRoyal as to product quality and workmanship. **DEROYAL'S WRITTEN WARRANTIES ARE GIVEN IN LIEU OF ANY IMPLIED WARRANTIES, INCLUDING WARRANTIES OF MERCHANTABILITY OR FITNESS FOR A PARTICULAR PURPOSE.**



Distributed By:

DeRoyal Industries, Inc.

200 DeBusk Lane

Powell, TN 37849 USA

888.938.7828 or (001) 865.938.7828

www.deroyal.com

PART#0-1884 | REVISED 2/2023

©2023 DEROYAL INDUSTRIES, INC.

All Rights Reserved. DeRoyal and the DeRoyal logo are trademarks or registered trademarks of

DeRoyal Industries, Inc.