### Explanation of Symbols

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### Explication des symboles

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### Verklaring van de symbolen

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migration of brace.
**STRAP 5:** Fasten anterior (front) strap and tighten securely.
**NOTE:** From side view, hinges should sit slightly posterior (behind) to midline of knee.
Adjustment of straps #1, #3, and #5 will influence this alignment. (See Image 3 & 4)
**4.** Stand up and take a few steps. Brace will drop slightly to align with center of kneecap. If adjustment is needed, hold brace to prevent rotation while straps are repositioned.

Application videos are available on www.DeRoyal.com.

#### OA CORRECTION

Your brace comes with a thin and thick condyle pad. Place the thin condyle pad on the affected side of your knee. Place the thick pad on the opposite side so that it provides correction.

#### TROUBLESHOOTING

It is normal for the brace to drop slightly after initial application so the hinge centers align with the center of the kneecap.

If the brace drops below knee center, reapply the brace using the following troubleshooting tips:

- STRAP 1:** Should rest on top of the calf muscle.
- STRAP 4:** Should not be too tight.
- STRAP 5:** Should be secure and snug, without restricting blood flow or causing discomfort. Tightening this strap also helps balance the pressure from the lower cuff across the tibia (shin).
- STRAP 1 AND 5:** Provide the most suspension control and may require adjustment to maintain proper suspension.
- Excess soft tissue, undefined calf muscles or unusual leg contour may require use of an undersleeve to help suspend your brace. Contact your orthotic professional for more information about undersleeves.

The brace comes with standard 5 degree extension stops. Additional flexion and extension stops may be installed if they are prescribed by your physician or needed for optimal performance or fit of the brace. These should be inserted by the technician who fits your brace.

#### WEAR TIME

Gradually increase brace wear time to allow your body to adjust to the brace. Start with 30 minutes on day one and gradually increase. Pain caused by osteoarthritis may take time to subside. It is important that you routinely wear the brace during weight bearing activities (e.g. when on your feet) as it will reduce joint loading and allow you to begin feeling relief. It may take a number of weeks for relief to become significant.

#### CLEANING AND/OR MAINTENANCE

- Salt water, sand, dirt and other elements can cause damage. Thoroughly rinse brace with fresh water after exposure to these conditions. Towel or air dry.
- Liners should be removed and hand washed as needed using cold water and mild detergent. Rinse thoroughly and air dry (Do not machine wash. Do not machine dry).
- Lubricate hinge with dry silicone spray as needed.

#### WARRANTY

DeRoyal products are warranted for one hundred twenty (120) days from the date of shipment from DeRoyal as to product quality and

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workmanship.
**DEROYAL'S WRITTEN WARRANTIES ARE GIVEN IN LIEU OF ANY IMPLIED WARRANTIES, INCLUDING WARRANTIES OF MERCHANTABILITY OR FITNESS FOR A PARTICULAR PURPOSE.**

**DeRoyal**<sup>®</sup>

Improving Care. Improving Business.™

## OA DOUBLE UPRIGHT KNEE BRACE

**RX ONLY**  
**SINGLE PATIENT USE**  
**NON-STERILE**  
**NOT MADE WITH NATURAL RUBBER LATEX**

**IMPORTANT INFORMATION**  
Please read all instructions, warnings, and precautions before use. Correct application is essential for proper functioning of the product.

Use only on the person it was provided to by a healthcare professional and only for the use it was intended.

**INTENDED USE**  
The DeRoyal<sup>®</sup> OA Double Upright Knee Brace is designed to help:

- Mild to severe unicompartamental knee osteoarthritis with ligament instabilities
- Increased rehabilitative and functional support for diagnoses that require reduced medial or lateral joint loads

**CONTRAINDICATIONS**  
Patients with poor peripheral circulation, diabetes or decreased skin sensitivity should not use this product.

#### PRECAUTIONS

- This product is to be fitted initially by a physician (or properly licensed practitioner) who is familiar with the purpose for which it is intended. The physician or practitioner is responsible for providing wearing instructions and precautions to other healthcare practitioners or healthcare providers involved in the patient’s care as well as the patient. Consult your physician (or properly licensed practitioner) immediately if you experience sensation changes, unusual reactions, swelling or increased pain while using this product. Follow the instructions of your physician (or properly licensed practitioner) for duration of use. These instructions do not supersede hospital protocol and/or direct orders of patient’s physician. Discontinue use of the device if it becomes damaged in any way, limiting its ability to perform as intended.
- This knee brace is designed to protect and reduce the risk of injury only. It is not intended to prevent or eliminate knee injuries.
- Flexion and extension stop adjustments to the hinge should be made only by, or under the advice of, your physician.

#### INSTRUCTIONS FOR USE

**NOTE:** These instructions assume that an orthopedic professional has already completed an initial fitting of this brace to your leg, and that strap lengths