UNIVERSAL DISPOSABLE SHOULDER IMMOBILIZER

	SINGLE PATIENT USE
	NON-Sterile
	NOT MADE WITH NATURAL RUBBER LATEX

RX ONLY FEDERAL U.S.A. LAW RESTRICTS THIS DEVICE TO SALE OR USE BY OR ON THE ORDER OF A PHYSICIAN OR PROPERLY LICENSED PRACTITIONER.

IMPORTANT INFORMATION

Please read all warnings and instructions before use. Correct application is essential for proper product function and to reduce the risk of injury or re-injury inherent with the use of any brace.

INTENDED USE

The Universal Sling Style Shoulder Immobilizer is used for S/P glenohumeral acromioplasty, surgical repair of the rotator cuff, anterior glenohumeral capsular reconstruction and anterior dislocation of the G-H joint.

CAUTIONS

- A physician or properly licensed practitioner (a "prescriber") who is familiar with the use and purpose of this brace must fit it to the user. The prescriber has a duty to provide wearing instructions and risks related to the use of this brace to other healthcare practitioners treating the users and the users themselves, including duration of use. The instructions provided in this sheet do not supersede hospital protocol or direct orders of the prescriber. Use only as directed.
- Prolonged usage can result in muscle atrophy in the user. The user may require (1) a weaning period from the brace to build muscle strength (2) regular physical therapy exercises to maintain the strength of the shoulder (3) a limitation on duration of use or (4) some combination of these.
- Inspect brace for damaged or missing components before use.
- Discontinue use and consult your prescriber if the brace or its components break or become damaged.
- Inspect the fit of the brace on the patient to ensure fit is proper. **DO NOT OVERTIGHTEN.** Check regularly to ensure circulation is not compromised.
- Consult your prescriber immediately if you experience sensation changes, unusual reactions, swelling or increased pain while using this brace. Discontinue use if pressure injuries develop.
- Wear clothing underneath this brace.
- Take special care if the user is diabetic or has poor circulation. These users may have decreased skin sensitivity and are at greater risk for poor peripheral circulation and pressure injuries.

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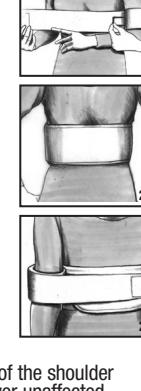
INSTRUCTIONS FOR USE


- Place body strap around waist and secure for a snug, but comfortable fit. Note: A shirt should be worn under this brace.
- Attach bicep cuff to body strap under affected arm with hook end toward the front, using attached piece of hook on the center of the bicep cuff. a) Wrap short end around arm toward the front and b) wrap long end around arm toward the back and secure onto body strap.
- Feed hook piece through the inside of the wrist cuff and secure to the center, front of the body strap.
- Secure shoulder strap to body strap behind unaffected shoulder using the hook end of the shoulder strap. Place shoulder strap over unaffected shoulder and feed strap through buckle on wrist cuff (placing the strap through the bottom of the buckle first).

CARE INSTRUCTIONS (IF APPLICABLE)

Hand wash, lukewarm water, mild detergent, air dry.

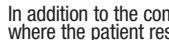STORAGE AND TRANSPORT CONDITIONS

In addition to the competent authority in the country where the patient resides, serious incidents must be reported to DeRoyal Industries, Inc.

WARRANTY

DeRoyal products are warranted for one hundred twenty (120) days from the date of shipment from DeRoyal as to product quality and workmanship.

DEROYAL'S WRITTEN WARRANTIES ARE GIVEN IN LIEU OF ANY IMPLIED WARRANTIES, INCLUDING WARRANTIES OF MERCHANTABILITY OR FITNESS FOR A PARTICULAR PURPOSE.

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