



Safety Auto-Reset™ Fall Monitor



INSTRUCTIONS FOR USE

SETTING UP THE MONITOR AND SENSOR PAD

1. Open battery compartment door with a small Phillips-head screwdriver. Insert batteries.
2. Close the battery door and secure it with the small Phillips-head screwdriver.
3. Place one bed sensor pad or chair sensor pad within reach.
4. Connect the monitor to the bed sensor pad or the chair sensor pad. Plug the cord end of the sensor pad into the bottom of the monitor. Pinch the cord together and route it through the cord protection recess on the monitor (near the plug input).
5. Put pressure on the sensor pad to activate the pad and monitor. The monitor will beep two times to indicate that the sensor pad and fall monitor are in use. The “Monitor Status” light will blink while pressure remains on the pad. When pressure is removed, the fall monitor will sound the alarm. Press the “Reset” button to silence the alarm.
6. If the “Pad Lost Light” is on, check to make sure the cord is plugged in and reset the system.
7. Test the system before use.

MOUNTING THE MONITOR

1. The strap with buckle can be used to attach the monitor to a desired location.
2. To mount the monitor, mount the screw onto the wall or headboard. Hang the monitor on the screw using the mounting hole on back of the monitor.

TURNING OFF OR RESETTING THE ALARM

Please refer to the images to the right for reference to 1 & 2.

1. When the alarm is triggered, the alarm can be turned off and reset with either the “Reset Mode” or “Caregiver Key™ Mode.”
2. Select either “Reset Button ON” or “Caregiver Key™ ON” with the switch on the side of the monitor. Use the end of a pen to move the switch.
3. If set on “Reset Button ON” position, the caregiver can reset the monitor by pressing the reset button located on the front of the monitor.
4. If set on the “Caregiver Key™ ON” position, the monitor can only be reset by using the Caregiver Key™ provided with the monitor. The Caregiver Key™ must be placed over the red dot to reset the monitor.

VOLUME, TONE AND DELAY OPTIONS

1. Adjust the volume by turning the dial located inside the battery compartment.
2. Select an alarm tone by adjusting the switch inside the battery compartment to the desired tone.
3. Select alarm tone delay time by adjusting the switch inside the battery compartment.

CARE INSTRUCTIONS

1. Wipe clean with disinfectant wipes.
2. After each cleaning, test device before use.

NOTE:

1. To disengage alarm when moving patient, place Caregiver Key™ over red dot marked Caregiver Key™. When patient re-engages alarm sensor, remove Caregiver Key™ to activate.
2. The Caregiver Key™ can be used in both “Reset Mode” and “Caregiver Key™ Mode” to disengage alarm.

