**SELF-RELEASING FOAM BELT**

SINGLE PATIENT USE	
	NON-Sterile
	NOT MADE WITH NATURAL RUBBER LATEX
RX ONLY	FEDERAL U.S.A. LAW RESTRICTS THIS DEVICE TO SALE OR USE BY OR ON THE ORDER OF A PHYSICIAN OR PROPERLY LICENSED PRACTITIONER.

**IMPORTANT INFORMATION**

Please read all instructions, warnings, and cautions before use. Correct application is essential for proper functioning of the product. Use only on the person it was provided to by a healthcare professional and only for the use it was intended.

**INTENDED USE**

DeRoyal® Self-Releasing Foam Belts are intended to be prescribed by a properly licensed practitioner to help prevent falls or getting up without assistance.

**CONTRAINDICATIONS**

- Do not use on a patient who is or may become highly aggressive or agitated.
- Never use on a patient whose I.V. or wound site could be compromised by the device.

**WARNINGS**

- Improper application or use of device may result in serious injury or death.
- Inspect the fit of the product on the patient to ensure fit is proper. **DO NOT OVERTIGHTEN.** Check regularly to ensure breathing and circulation are not compromised.

**CAUTIONS**

- A physician or properly licensed practitioner (a "prescriber") who is familiar with the use and purpose of this product must fit it to the user. The prescriber has a duty to provide wearing instructions and risks related to the use of this product to other healthcare practitioners treating the users and the users themselves, including duration of use. The instructions provided in this sheet do not supersede hospital protocol or direct orders of the prescriber.
- Inspect product for damaged or missing components before use.
- Discontinue use and consult your prescriber if the product or its components break or become damaged, or cleaning is no longer sufficient to sanitize product.
- Belt should be worn over patient's clothing. Consult prescriber immediately if patient experiences sensation changes, unusual reactions, swelling or increased pain while using this product. Discontinue use if pressure injuries develop.
- Take special care if the user is diabetic or has poor circulation. These users may have decreased skin sensitivity and are at greater risk for poor peripheral circulation and pressure injuries.

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**INSTRUCTIONS FOR USE****APPLICATION TO WHEELCHAIR OR RECLINER**

- Wrap the belt around the back of the chair and bring forward to front of the patient. Secure hook and loop closure in front of the patient. **NOTE:** For a more restrictive device, make sure the hook and loop closure is behind the back of the chair.
- Do not over tighten. Two fingers should fit between the device and the patient to ensure breathing and circulation is not compromised.

**APPLICATION TO BED**

- Patient should be sitting up while belt is being applied. Wrap the belt around the portion of the bed frame that moves with the patient and bring forward to front of the patient. Secure hook and loop closure in front of the patient. Ensure belt is applied for a snug fit. **NOTE:** For a more restrictive device, make sure the hook and loop closure is under the bed frame that moves with the patient.
- Do not over-tighten. Two fingers should fit between the device and the patient to ensure that breathing and circulation are not compromised.

**CLEANING AND/OR MAINTENANCE**

Hand wash with a mild detergent in lukewarm water. Air dry prior to wearing. Inspect the product after washing to ensure it is intact. If the product is damaged or cleaning is insufficient, do not use it and replace it with a new one.

**STORAGE AND TRANSPORT CONDITIONS**

	KEEP DRY
	KEEP AWAY FROM SUNLIGHT

In addition to the competent authority in the country where the patient resides, serious incidents must be reported to DeRoyal Industries, Inc.

**WARRANTY**

DeRoyal® products are warranted for one hundred twenty (120) days from the date of shipment from DeRoyal as to product quality and workmanship. **DEROYAL'S WRITTEN WARRANTIES ARE GIVEN IN LIEU OF ANY IMPLIED WARRANTIES, INCLUDING WARRANTIES OF MERCHANTABILITY OR FITNESS FOR A PARTICULAR PURPOSE.**

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