




Improving Care. Improving Business.®

LEATHER SECURITY CUFFS

	SINGLE PATIENT USE
	NON-STERILE
	NOT MADE WITH NATURAL RUBBER LATEX
RX ONLY	FEDERAL U.S.A. LAW RESTRICTS THIS DEVICE TO SALE OR USE BY OR ON THE ORDER OF A PHYSICIAN OR PROPERLY LICENSED PRACTITIONER.

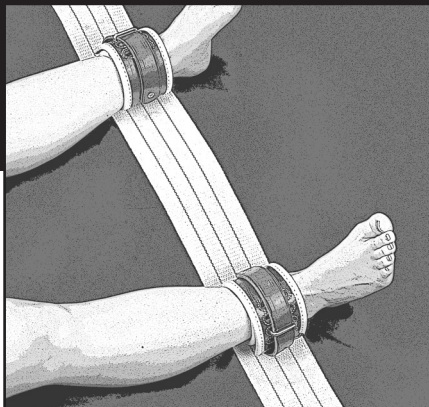

IMPORTANT INFORMATION

Please read all instructions, warnings, and cautions before use. Correct application is essential for proper functioning of the product. Use only on the person it was provided to by a healthcare professional and only for the use it was intended.

INTENDED USE

The DeRoyal® Security Cuffs are intended to be prescribed by a properly licensed practitioner to protect the patient from harming themselves, other patients, family members and/or staff.

CONTRAINDICATIONS

Never use on a patient if an I.V. or wound site could be compromised by the device.

WARNINGS

- Keep straps and buckles out of patient's reach. Patient should not be able to remove device. Improper application or use of any restraint may result in serious injury or death.
- Inspect product for damage, missing components, or contamination before use. Ensure that all hook and loop and buckle closures function properly.
- Use hospital approved bed safety guidelines. If securing the device to a bed always secure straps to the portion of the bed that moves with the patient. Do not secure straps to bed side rails.
- Check the patient regularly per hospital's policies and procedures to ensure circulation is not compromised.

CAUTIONS

- This product is to be fitted initially by a physician (or properly licensed practitioner) who is familiar with the purpose for which it is intended. The physician or practitioner is responsible for providing wearing instructions and cautions to other healthcare practitioners or healthcare providers involved in the patient's care.
- Consult physician (or properly licensed practitioner) immediately if patient experiences sensation changes, unusual reactions, swelling or increased pain while using this device. Be sure to follow your facility's policies and guidelines for frequency of patient monitoring.

INSTRUCTION FOR USE FIXED POSITION CUFFS

1. Lay the device horizontally across the bed, approximately where the patient's wrists or ankles will be resting.
2. Find a location on the bed frame that moves with the patient as specified by hospital bed safety guidelines.
3. Unfasten the buckles from one end of the strap and wrap it around a single location on the portion of the bed that moves with the patient.
4. Secure the buckle, by feeding the end of the strap through the metal ring and inserting the pin into the hole of the strap (like a belt buckle).
NOTE: Make sure the strap is secured as tight as possible to the portion of the bed that moves with the patient.
5. Open each cuff and secure the cuffs to the wrist or ankle.
6. Wrap the cuff around the wrists or ankles with padded side touching the patient's skin.
7. Secure the cuff closure by bringing the metal ring to the closest vertical slot on the cuff, then insert the strap through the metal ring to secure.
NOTE: One finger should fit between the device and the patient

CUFFS WITH CONNECTING STRAPS

1. Open the cuff and secure to the wrist or ankle.
2. Wrap the cuff around the wrists or ankles with padded side touching the patient's skin.
3. Secure the cuff closure by bringing the metal ring to the closest vertical slot on the cuff, then insert the strap through the metal ring to secure.
NOTE: One finger should fit between the device and the patient
4. Find a location on the bed frame that moves with the patient as specified by hospital bed safety guidelines.
5. Unfasten the buckle and feed the end of the strap into the metal ring on the cuff. Take the end of the strap and wrap it around a single location on the portion of the bed that moves with the patient.
6. Secure the buckle, by feeding the end of the strap through the metal ring and inserting the pin into the hole of the strap (like a belt buckle).
NOTE: Make sure the strap is secured as tight as possible to the portion of the bed that moves with the patient.



Improving Care. Improving Business.®

CLEANING AND/OR MAINTENANCE

To remove debris, wipe with mild soap and warm water. When disinfecting, dilute one cup of bleach with a gallon of water. Take damp cloth along with the diluted solution to wipe down contaminated area. Allow it to air dry and apply leather conditioner as needed.

STORAGE AND TRANSPORT CONDITIONS

	KEEP DRY
	KEEP AWAY FROM SUNLIGHT

WARRANTY

DEROYAL® PRODUCTS ARE WARRANTED FOR NINETY (90) DAYS FROM THE DATE OF SHIPMENT FROM DEROYAL AS TO PRODUCT QUALITY AND WORKMANSHIP. **DEROYAL'S WRITTEN WARRANTIES ARE GIVEN IN LIEU OF ANY IMPLIED WARRANTIES, INCLUDING WARRANTIES OF MERCHANTABILITY OR FITNESS FOR A PARTICULAR PURPOSE.**



Improving Care. Improving Business.®



Manufactured by:
DeRoyal Industries, Inc.
200 DeBusk Lane
Powell, TN 37849 USA
888.938.7828 or (001) 865.938.7828
www.deroyal.com

Part#0-719 Revised 6/2021

©2021 DeRoyal Industries, Inc.

All Rights Reserved. DeRoyal, the DeRoyal logo, and Improving Care. Improving Business. are trademarks or registered trademarks of DeRoyal Industries, Inc.