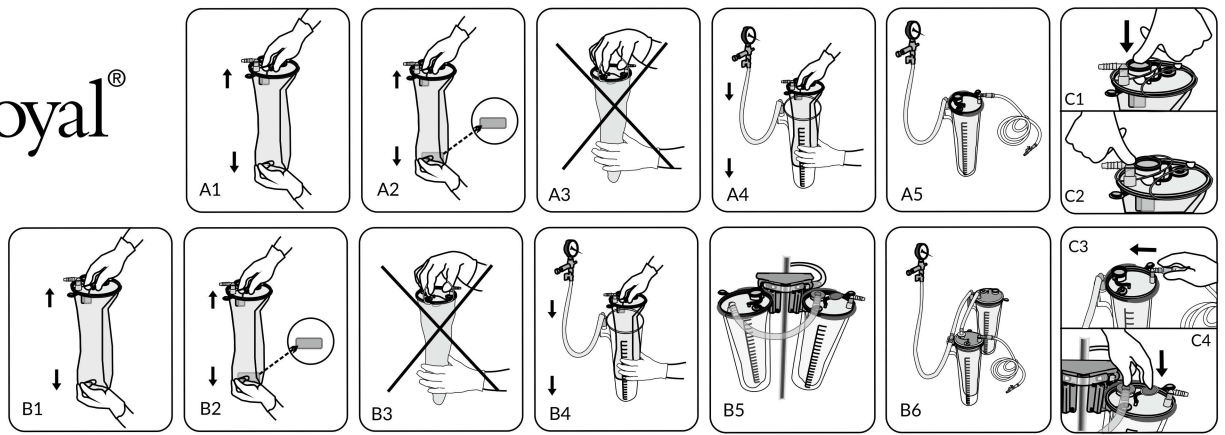
EN INSTRUCTION FOR INSTALLATION OF SUCTION LINER

INTENDED USE

The DeRoyal Disposable Suction System is used for hygienic suction of various fluids and / or secretion from patients during medical treatment. The suction system has no contact with the human body. For single connections of DeRoyal-Suction Liner please follow the instructions according to Figures A1-A5 and C1-C3.

For installation of serial connections please see Figures B1-B6 and C1-C3.

Recommended application for solidifying agent (With Sample Port Models);

Dosage is dependent on the composition of the fluids. A high percentage of electrolytes in the solution typically requires a higher dosage. It is recommended to add extra solidifying agent if necessary.

STORAGE AND TRANSPORTATION CONDITIONS

Products should be stored in a closed warm and dry environment. All products should be protected from damp, dirty and dusty environments. Products can be transported under ambient atmospheric conditions provided that they are protected from rain, dust and direct sunlight. Temperature-controlled transportation is not required.

DISPOSAL

All disposable products used in the DeRoyal Disposable Suction System should be replaced for each patient. If long-term treatment is to be performed on a single patient, it is advisable to change the disposable products within 24 hours but at the latest after 72 hours. The products should be disposed in accordance with hospital disposal policy. **Turn off the vacuum after the suction procedure.**

SHELF LIFE

Shelf life of DeRoyal-Suction Liner is 5 years. Shelf life of DeRoyal-Suction Liner with solidifying agent is 4 years.

WARNINGS

The DeRoyal Disposable Suction System is only to be used by qualified healthcare professionals trained for the use of the suction system. Do not use any component of the system if there is any apparent damage. DeRoyal-Suction Liner must only be used with the same size of DeRoyal Canisters. Close all the ports before turning off the vacuum. Make sure connections are done correctly, according to Figure A or B series. Make sure that all the connections are securely fitted and pressed firmly in place before starting operation, according to Figure C series. Do not open the vertical fold.

Do not remove the suction liner before the vacuum reaches zero.

There is a risk of cracking of the lid if the suction liner is left under high vacuum for a long time. If using the suction liner, the vacuum source is recommended to be turned off if the procedure is not shortly started. Check that the product is intact before starting suction. Failure to do so can delay starting of the critical operation.

Do not exceed vacuum level of 736 mmHg (1 bar).

Do not leave the system under high vacuum for more than 1 hour while the suction tubing is closed using a stop vacuum adapter or a push-activated vacuum stop connector.

Do not attempt to move the trolley by pulling on the patient tube and do not pull on the patient tube. The patient connector may become detached from the patient tube.

DeRoyal uses the symbols and meanings from standards ISO 15523-1. These symbols are placed next to the text explaining their meaning in this instructions for use (IFU), a complete symbols glossary is available online at deroyal.com/symbols or by contacting customer service.

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DeRoyal Industries, Inc.