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WARRIOR® ELBOW POST-OPERATIVE BRACE

SINGLE PATIENT USE
NON-Sterile
NOT MADE WITH NATURAL RUBBER LATEX
RX ONLY

FEDERAL U.S.A. LAW RESTRICTS THIS DEVICE TO SALE OR USE BY OR ON THE ORDER OF A PHYSICIAN OR PROPERLY LICENSED PRACTITIONER.

IMPORTANT INFORMATION

Please read all warnings and instructions before use. Correct application is essential for proper product function and to reduce the risk of injury or re-injury inherent with the use of any brace.

INTENDED USE

The DeRoyal® Warrior® Elbow Post-Operative Brace is designed to provide elbow immobilization or range of motion limitations as prescribed by a clinician. It may be used post injury, post-operatively or throughout rehabilitation.

WARNING

- Inspect brace for damaged or missing components prior to patient application.
- Discontinue use and consult your prescriber if the brace or its components break or become damaged.

CAUTIONS

- A physician or properly licensed practitioner (a "prescriber") who is familiar with the use and purpose of this brace must fit it to the user. The prescriber has a duty to provide wearing instructions and risks related to the use of this brace to other healthcare practitioners treating the users and the users themselves, including duration of use. The instructions provided in this sheet do not supersede hospital protocol or direct orders of the prescriber.
- Prolonged usage can result in muscle atrophy in the user. The user may require (1) a weaning period from the brace to build muscle strength (2) regular physical therapy exercises to maintain strength (3) a limitation on duration of use, or (4) some combination of these.
- Inspect the fit of the brace on the patient to ensure fit is proper. DO NOT OVERTIGHTEN. Check regularly to ensure circulation is not compromised.
- Consult your prescriber immediately if you experience sensation changes, unusual reactions, swelling or increased pain while using this brace. Discontinue use if pressure injuries develop.
- Take special care if the user is diabetic or has poor circulation at the patient's site of use.

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circulation as these users may have decreased skin sensitivity and are at greater risk for poor peripheral circulation and pressure injuries.

INSTRUCTIONS FOR USE

BRACE APPLICATION:

- Unfasten all hook and loop straps.
- Position the brace laterally and align the hinge of the brace with the elbow joint. Brace should fit comfortably up the bicep/underarm and out to the patient's wrist (see Length Adjustment).
- Fixate the wrist pad by shaping it around the patient's wrist.
- Beginning with the forearm strap, fasten all four hook and loop straps.

CLEANING INSTRUCTIONS

Hand wash liner pads with mild detergent in lukewarm water. Air dry prior to wearing.

STORAGE AND TRANSPORT CONDITIONS

	KEEP DRY
	KEEP AWAY FROM SUNLIGHT

In addition to the competent authority in the country where the patient resides, serious incidents must be reported to DeRoyal Industries, Inc.

LENGTH ADJUSTMENT:

- Unlock length adjustment levers located at the top and bottom of the brace (blue lever).
- Adjust to accommodate bicep and forearm length by sliding the telescoping bars.
- Lock length adjustment levers securely in the lock position (indicated by lock and unlock icons on brace).

FLEXION/EXTENSION ADJUSTMENT:

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All Rights Reserved. Warrior, DeRoyal, the
Business, are trademarks or registered
trademarks of DeRoyal Industries, Inc.