PUCCI® EZE HAND ORTHOSIS

ORTESIS DE MANO PUCCI® EZE

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PUCCI® EZE HANDORTHESE

PUCCI® EZE ORTOSI PER MANO

ÓRTESE PARA MÃO PUCCI® EZE

PUCCI® EZE-HANDORTHESE

PUCCI® EZE HANDORTOS

PUCCI® EZE EL ORTEZI

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PUCCI® EZE HAND ORTHOSIS

	SINGLE PATIENT USE
	NON-STERILE
	NOT MADE WITH NATURAL RUBBER LATEX
	FEDERAL U.S.A. LAW RESTRICTS THIS DEVICE TO SALE OR USE BY OR ON THE ORDER OF A PHYSICIAN OR PROPERLY LICENSED PRACTITIONER.
	RX ONLY

IMPORTANT INFORMATION

Please read all warnings and instructions before use. Correct application is essential for proper product function and to reduce the risk of injury or re-injury inherent with the use of any brace.

INTENDED USE

The DeRoyal® Pucci® EZE Hand Orthosis is intended to provide continuous therapy to the contracted fingers, hand and wrist.

CONTRAINDICATIONS

Severe or fixed contractures; excessive edema or swelling; excessive pain.

CAUTIONS

- A physician or properly licensed practitioner (a “prescriber”) who is familiar with the use and purpose of this brace must fit it to the user. The prescriber has a duty to provide wearing instructions and risks related to the use of this product to other healthcare practitioners treating the users and the users themselves, including duration of use. The instructions provided in this sheet do not supersede hospital protocol or direct orders of the prescriber.
- Use only as directed by medical professional.
- Prolonged usage can result in muscle atrophy in the user. The user may require (1) a weaning period from the brace to build muscle strength (2) regular physical therapy exercises to maintain the strength of the hand and wrist (3) a limitation on duration of use or (4) some combination of these.
- Inspect brace for damaged or missing components before use.
- Discontinue use and consult your prescriber if the brace or its components break or become damaged.
- Inspect the fit of the brace on the patient to ensure fit is proper. **DO NOT OVERTIGHTEN.** Check regularly to ensure circulation is not compromised.
- Consult your prescriber immediately if you experience sensation changes, unusual reactions, swelling or increased pain while using this brace. Discontinue use if pressure injuries develop.
- Take special care if the user is diabetic or has poor circulation as these users may have decreased skin sensitivity and are at greater risk for poor peripheral circulation and pressure injuries.

INSTRUCTIONS FOR USE

- The Pucci® EZE Hand Orthosis is supplied with a small and large hand roll. The orthosis can be used without a hand roll, or with either the small or large hand roll.
- Determine the angle of the wrist position. Form the orthosis to conform to the wrist.
- Without a hand roll, carefully open the fingers and position the palmar section of the orthosis under the fingers.
- Fasten the hook and loop closures over the forearm, wrist and

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fingers. Check that they are secure and allow adequate circulation.

- Check the fit of the entire orthosis, and adjust, if necessary, to ensure proper fit. Bend the wrist portion into a new position as needed to increase wrist range of motion.
- The hand rolls may be added to the Pucci® EZE Orthosis for static progressive positioning. Insert the hand roll into the hand roll cover.
- Insert the loop strap on the thumb side of the orthosis through the opening in the hand roll. Slide the hand roll onto the palmar section of the orthosis.
- Apply the orthosis with the fingers wrapped around the the hand roll.
- Once the orthosis is fitted, check the fit. Follow your facilities protocol. If no protocol is provided by your facility, check the fit of the orthosis at increasingly longer intervals of time. Start with the orthosis on for 15 minutes and off for 15 minutes. **DO NOT** exceed seven hours on at any time.
- Recheck the fingers and thumb for desired position over the palmar section or hand roll. Assess fingers for adequate circulation. If circulation is impaired, remove orthosis immediately.

REPLACING COVERS

Slide the cover off the forearm support and palmar section. Slide the cover off the palmar addition. Reapply the cover to the main frame of the splint by sliding the cover over the forearm support and palmar section of the splint. If using the hand roll, slide the hand roll cover of the splint over the hand roll frame.

CARE INSTRUCTIONS

FRAME

Hand wash in lukewarm water with a mild detergent. Air dry prior to wearing.

COVER

Remove the cover and fasten all hook and loop closures. Place in mesh laundry bag. Machine wash on gentle cycle with mild detergent and warm water. **DO NOT** use bleach or fabric softener. Rinse in cold water and air dry prior to wearing. Covers can be washed but not sterilized.

STORAGE AND TRANSPORT CONDITIONS

	KEEP DRY
	KEEP AWAY FROM SUNLIGHT

In addition to the competent authority in the country where the patient resides, serious incidents must be reported to DeRoyal Industries, Inc.

WARRANTY

DeRoyal® products are warranted for one hundred twenty (120) days from the date of shipment from DeRoyal as to product quality and workmanship. **DEROYALS WRITTEN WARRANTIES ARE GIVEN IN LIEU OF ANY IMPLIED WARRANTIES, INCLUDING WARRANTIES OF MERCHANTABILITY OR FITNESS FOR A PARTICULAR PURPOSE.**

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All Rights Reserved. DeRoyal, the DeRoyal logo, Improving Care. Improving Business. and Pucci are trademarks or registered trademarks of DeRoyal Industries, Inc.