EN: PUCCI® AIR INFLATABLE HAND ORTHOSIS
ES: ORTESIS DE MANO INFLABLE PUCCI®
FR: ORTHÈSE GONFLABLE POUR LA MAIN PUCCI® AIR
DE: PUCCI® AIR AUFBLASBARE HANDORTHESE
IT: PUCCI® AIR ORTOSI GONFIABILE PER MANO
PT: ÓRTESE INFLÁVEL PARA MÃO PUCCI® AIR
NL: PUCCI® AIR-OPBLAASBARE HANDORTHSE
SE: PUCCI® AIR UPPBLÅSBAR HANDORTOS
TR: PUCCI® HAVALI ŞİŞİRİLEBİLİR EL ORTEZİ
EL: PUCCI® AIR ΦΟΥΣΚΩΤΗ ΟΡΟΣΗ ΧΕΡΙΟΥ

Manufactured by:
DeRoyal Industries, Inc.
200 DeBusk Lane
Powell, TN 37849 USA
888.938.7828 or (001) 865.938.7828
www.deroyal.com

EC REP
Prinsesgracht 20
2514 AP The Hague
The Netherlands

Part#1-413G

Revised 8/22

©2022 DeRoyal Industries, Inc.
All Rights Reserved. DeRoyal, the DeRoyal logo, Improving Care, Improving Business, and Pucci are trademarks or registered trademarks of DeRoyal Industries, Inc.

PUCCI® AIR INFLATABLE HAND ORTHOSIS

	SINGLE PATIENT USE
	NON-Sterile
	NOT MADE WITH NATURAL RUBBER LATEX
RX ONLY	FEDERAL U.S.A. LAW RESTRICTS THIS DEVICE TO SALE OR USE BY OR ON THE ORDER OF A PHYSICIAN OR PROPERLY LICENSED PRACTITIONER.

IMPORTANT INFORMATION

Please read all warnings and instructions before use. Correct application is essential for proper product function and to reduce the risk of injury or re-injury inherent with the use of any brace.

INTENDED USE

The DeRoyal® Pucci® Air Inflatable Hand Orthosis is intended to provide continuous low stretch therapy to the contracted hand.

CONTRAINDICATIONS

Severe or fixed contractures; excessive edema or swelling; excessive pain.

CAUTIONS

- A physician or properly licensed practitioner (a "prescriber") who is familiar with the use and purpose of this brace must fit it to the user. The prescriber has a duty to provide wearing instructions and risks related to the use of this product to other healthcare practitioners treating the users and the users themselves, including duration of use. The instructions provided in this sheet do not supersede hospital protocol or direct orders of the prescriber.
- Use only as directed by medical professional.
- Prolonged usage can result in muscle atrophy in the user. The user may require (1) a weaning period from the brace to build muscle strength (2) regular physical therapy exercises to maintain the strength of the hand and wrist (3) a limitation on duration of use or (4) some combination of these.
- Inspect brace for damaged or missing components before use.
- Discontinue use and consult your prescriber if the brace or its components break or become damaged.
- Inspect the fit of the brace on the patient to ensure fit is proper. **DO NOT OVERTIGHTEN.** Check regularly to ensure circulation is not compromised.
- Consult your prescriber immediately if you experience sensation changes, unusual reactions, swelling or increased pain while using this brace. Discontinue use if pressure injuries develop.
- Take special care if the user is diabetic or has poor circulation as these users may have decreased skin sensitivity and are at greater risk for poor peripheral circulation and pressure injuries.

INSTRUCTIONS FOR USE

- Determine the degree of wrist position. If the wrist position is neutral, or 0° to 15° of extension, apply the Pucci® Air orthosis as it comes from the package. If the flexion contracture is 5° or more, bend the orthosis at the narrow

portion, below the bladder, to conform to the wrist.

- Insert the **BLACK END** of the inflation/deflation bulb into the air valve on the orthosis, and squeeze the bulb to deflate the bladder.
- Open all hook and loop closures on the orthosis.
- Position the air valve at the little finger side of the hand and carefully slip the deflated bladder under the fingers as far as comfort will allow.
- Gently position the thumb around the bladder.
- Fasten the hook and loop closures straps over the wrist and forearm. Check that the straps are secure and allow adequate circulation.
- Inflate the bladder per physician's order by inserting the **WHITE END** of the inflation/deflation bulb into the air valve on the orthosis, and squeezing.
- After inflation, fasten the hook and loop closure straps over the hand or fingers to maintain the bladder in the correct position under the fingers and thumb.
- Recheck the fingers and thumb for desired position over the inflated bladder. Assess fingers for adequate circulation. If circulation is impaired, remove orthosis immediately.
- To remove the orthosis, deflate the bladder, and open all hook and loop closure straps.

REPLACING COVERS

To remove the cover, deflate the bladder. Slide the frame out of the cover through the opening on the back. To replace the liner, insert the deflated frame into the clean cover, lining up the inflation valve with the small opening in the cover.

CARE INSTRUCTIONS
BLADDER, INFLATION BULB AND WRIST SUPPORT

Hand wash in lukewarm water with a mild detergent. Air dry prior to wearing.

COVER

Remove the liner and fasten all hook and loop closures. Place in mild laundry bag. Machine wash on gentle cycle with mild detergent and warm water. **DO NOT** use bleach or fabric softener. Rinse in cold water and air dry. Covers can be washed but not sterilized.

STORAGE AND TRANSPORT CONDITIONS

	KEEP DRY
	KEEP AWAY FROM SUNLIGHT

In addition to the competent authority in the country where the patient resides, serious incidents must be reported to DeRoyal Industries, Inc.

WARRANTY

DeRoyal® products are warranted for one hundred twenty (120) days from the date of shipment from DeRoyal as to product quality and workmanship. **DEROYAL'S WRITTEN WARRANTIES ARE GIVEN IN LIEU OF ANY IMPLIED WARRANTIES, INCLUDING WARRANTIES OF MERCHANTABILITY OR FITNESS FOR A PARTICULAR PURPOSE.**

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