PUCCI® FINGER SEPARATOR

	SINGLE PATIENT USE
	NON-Sterile
	NOT MADE WITH NATURAL RUBBER LATEX
	RX ONLY FEDERAL U.S.A. LAW RESTRICTS THIS DEVICE TO SALE OR USE BY OR ON THE ORDER OF A PHYSICIAN OR PROPERLY LICENSED PRACTITIONER.

IMPORTANT INFORMATION

Please read all warnings and instructions before use. Correct application is essential for proper product function and to reduce the risk of injury or re-injury inherent with the use of any brace.

INTENDED USE

The DeRoyal® Pucci® Finger Separator is intended to help position mild to moderate contracted or deviated fingers.

CAUTIONS

- A physician or properly licensed practitioner (a "prescriber") who is familiar with the use and purpose of this brace must fit it to the user. The prescriber has a duty to provide wearing instructions and risks related to the use of this product to other healthcare practitioners treating the users and the users themselves, including duration of use. The instructions provided in this sheet do not supersede hospital protocol or direct orders from the prescriber.
- Use only as directed by medical professional.
- Prolonged usage can result in muscle atrophy in the user. The user may require (1) a weaning period from the brace to build muscle strength (2) regular physical therapy exercises to maintain the strength of the fingers (3) a limitation on duration of use or (4) some combination of these.
- Inspect brace for damaged or missing components before use.
- Discontinue use and consult your prescriber if the brace or its components break or become damaged.
- Inspect the fit of the product on the patient to ensure fit is proper. **DO NOT OVERTIGHTEN.** Check regularly to ensure circulation is not compromised.
- Consult your prescriber immediately if you experience sensation changes, unusual reactions, swelling or increased pain while using this brace. Discontinue use if pressure injuries develop.
- Take special care if the user is diabetic or has poor circulation as these users may have decreased skin sensitivity and are at greater risk for poor peripheral circulation and pressure injuries.

INSTRUCTIONS FOR USE

- Open the hook and loop closure.
- Carefully open the fingers and insert them into the loops of the finger separator, and position. **NOTE:** Do not put the thumb into a loop of the finger separator.
- Cross the straps on the palmar side of the hand, wrap around and fasten together on the dorsal side of the hand, across the metacarpophalangeal (MP) joints.
- If using the finger separator with a hand orthosis, apply using steps 1 – 3. Apply the hand orthosis. Wrap the straps of the finger separator around the palmar piece of the hand orthosis and fasten with the hook and loop closures.
- Once the orthosis is fitted, check the fit. Follow your facilities protocol. If no protocol is provided by your facility, check the fit of the orthosis at increasingly longer intervals of time. Start with the orthosis on for 15 minutes and off for 15 minutes. **DO NOT** exceed seven hours on at any time.
- Recheck the fingers for the desired position. Assess fingers for adequate circulation. If circulation is impaired, remove orthosis immediately.

CARE INSTRUCTIONS

Hand wash in lukewarm water with a mild detergent. Air dry prior to wearing.

STORAGE AND TRANSPORT CONDITIONS

	KEEP DRY
	KEEP AWAY FROM SUNLIGHT

In addition to the competent authority in the country where the patient resides, serious incidents must be reported to DeRoyal Industries, Inc.

WARRANTY

DeRoyal® products are warranted for one hundred twenty (120) days from the date of shipment from DeRoyal as to product quality and workmanship. **DEROYAL'S WRITTEN WARRANTIES ARE GIVEN IN LIEU OF ANY IMPLIED WARRANTIES, INCLUDING WARRANTIES OF MERCHANTABILITY OR FITNESS FOR A PARTICULAR PURPOSE.**

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**Part#1-812G**

Revised 2/2021

©2021 DeRoyal Industries, Inc.
All Rights Reserved. DeRoyal, the DeRoyal logo, Improving Care, Improving Business, and Pucci are trademarks or registered trademarks of DeRoyal Industries, Inc.

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