Explanation of Symbols

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Explication des symboles

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Verklaring van de symbolen

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- tip from Solidifier caps.
- Pour Solidifier into suction canister using the scale on the side of the bottle to determine how much Solidifier to use.
 4. Fill DeRoyal's Solidifier to gel.
 5. Fast-acting polymers do all of the work, no shaking required.
 6. Re-cap multi-use bottle using blue tip provided.
 7. Dispose of waste in accordance with Federal, State and Local regulations.

WARRANTY

DeRoyal products are warranted for one hundred twenty (120) days from the date of shipment from DeRoyal as to product quality and workmanship. **DEROYAL'S WRITTEN WARRANTIES ARE GIVEN IN LIEU OF ANY IMPLIED WARRANTIES, INCLUDING WARRANTIES OF MERCHANTABILITY OR FITNESS FOR A PARTICULAR PURPOSE.**

- Read entire label and SDS prior to use of this product.
- Follow facility protocols regarding disposal. Follow regulations regarding proper disposal of biohazard material.
- Solidifier spill may create a slippery surface. Follow SDS regarding spill clean up.

SOLIDIFIER

SINGLE PATIENT USE ONLY
DISPOSABLE
NOT MADE WITH NATURAL RUBBER LATEX

IMPORTANT INFORMATION

Please read all instructions, warnings, and precautions before use. Correct application is essential for proper functioning of product.

INTENDED USE

DeRoyal Solidifier is used to quickly and completely solidify blood and body fluids for disposal.

INDICATIONS

This solidifier is for use in surgery and emergency departments, special care units, and patient care rooms to facilitate the safe handling of liquid waste.

WARNINGS

- Read entire label and SDS prior to use of this product.
- Follow facility protocols regarding disposal. Follow regulations regarding proper disposal of biohazard material.
- Solidifier spill may create a slippery surface. Follow SDS regarding spill clean up.

CAUTIONS

- Do not inhale or ingest product.
- Before use, check packaging for any damage and product for any contaminate or clumping.

DIRECTIONS

DISSOLVABLE POUCHES

- Practice universal precautions. Wear gloves and eye protection when handling infectious waste.
- Open pour spout on suction canister.
- Place entire pouch inside canister. This solidifier is a dissolvable pouch. It is not necessary to open the pouch.
- Close pour spout and allow DeRoyal's Solidifier to gel.
- Fast-acting polymers do all of the work, no shaking required.
- Dispose of waste in accordance with federal, state and local regulations.

SINGLE-USE BOTTLES

- Practice universal precautions. Wear gloves and eye protection when handling infectious waste.
- Open pour spout on suction canister.
- Unscrew lid on Solidifier bottle or remove blue tip from Solidifier cap.
- Pour Solidifier into canister and allow DeRoyal's Solidifier to gel.
- Fast-acting polymers do all of the work, no shaking required.
- Dispose of waste in accordance with federal, state and local regulations.

MULTI-USE BOTTLES

- Practice universal precautions. Wear gloves and eye protection when handling infectious waste.
- Open pour spout on suction canister and remove blue

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All Rights Reserved.